



Guide to Component Selection

Step 1 | Select system size, unilateral or bilateral, by patient weight:

Triple Action Weight Limits (kg/lb)

System Size	Unilateral	Bilateral
3A76 (19mm)	100 kg / 220 lb	110 kg / 240 lb
3B76 (16mm)	90 kg / 200 lb	100 kg / 220 lb
3C76* (13mm)	40 kg / 90 lb	50 kg / 110 lb

*Requires Companion Joint

Step 2 | Select booster configuration and order number by gait deficit

GAIT PHASE WHERE ABNORMALITIES OCCUR	Early Stance Phase				NORMAL	ABNORMAL				NORMAL				ABNORMAL								
	Late Stance Phase					NORMAL				ABNORMAL												
Booster Configuration					No Booster					Plantarflexion Resist Booster					Dorsiflexion Resist Booster							Plantarflexion & Dorsiflexion Resist Boosters
Leg	Right	Left	Left	Right	Either	Right	Left	Left	Right	Right	Left	Left	Right	Right	Left	Left	Right	Right	Left	Left	Right	Either
Side	Lateral	Medial	Lateral	Medial		Lateral	Medial	Lateral	Medial	Lateral	Medial	Lateral	Medial	Lateral	Medial	Lateral	Medial	Lateral	Medial	Lateral	Medial	
Order No.	3A76-A1		3A76-A0		3B76-A0	3A76-A3		3A76-A2		3A76-A1*		3A76-A0*		3A76-A3		3A76-A2		3B76-A3				
					3C76-A0	3B76-A1		3B76-A2		3B76-A1		3C76-A3										
						3C76-A1		3C76-A2		3C76-A1												

Patent Pending
©2020 Becker Orthopedic Appliance Co.,
All rights reserved.
Revision 05/01/20



F 800-521-2192 | 248-588-7480
P 800-923-2537 | 248-588-2960
BeckerOrthopedic.com | f t in

EC REP

Acorn Regulatory Consultancy
Services Limited
Knockmorris Cahir Co. Tipperary
Ireland, Postcode: E21 R766
P 012 4626 8456
F 012 4626 8648



Step 3 | Select fabrication method:

System Size	Thermoplastic	Composite
3A76	 Unilateral* 5mm (3/16") Polypropylene	 Unilateral Wet Lamination like Becker BCO Shadow™, Max™ or PrePreg
3B76	 Bilateral 5mm (3/16") Polypropylene	 Bilateral Wet Lamination like Becker BCO Lite™, Shadow™, Max™ or PrePreg
3C76	 Unilateral* 4mm (5/32") Polypropylene	 Bilateral 4mm (5/32") Polypropylene

Step 4 | Select stirrup:

3A76											
											
Left	Right	Left	Right	Left	Right	Left	Right	Left	Right	Left	Right
Lateral		Medial		Lateral		Medial		Lateral		Medial	
3A76-LATL-1		3A76-LATR-1		3A76-MEDL-1		3A76-MEDR-1		3A76-YL-1		3A76-YR-1	
								3A76-YL-1		3A76-YR-1	

3B76											
											
Left	Right	Left	Right	Left	Right	Left	Right	Left	Right	Left	Right
Lateral		Medial		Lateral		Medial		Lateral		Medial	
3B76-LATL-1		3B76-LATR-1		3B76-MEDL-1		3B76-MEDR-1		3B76-Y-1		3B76-R-1	
								3B76-Y-1		3B76-R-1	

3C76											
											
Left	Right	Left	Right	Left	Right	Left	Right	Left	Right	Left	Right
Lateral		Medial		Lateral		Medial		Lateral		Medial	
3C76-LAT-1		3C76-MEDL-1		3C76-MEDR-1		3C76-YLNG-1		3C76-R-1			
								3C76-R-1			

Patent Pending
©2020 Becker Orthopedic Appliance Co.,
All rights reserved.
Revision 05/01/20



F 800-521-2192 | 248-588-7480
P 800-923-2537 | 248-588-2960
BeckerOrthopedic.com | f t in

EC REP
Acorn Regulatory Consultancy
Services Limited
Knockmorris Cahir Co. Tipperary
Ireland, Postcode: E21 R766
P 012 4626 8456
F 012 4626 8648

