



# How to fit the Paww

A step-by-step guide to fitting the Paww for users with partial hand or finger limb differences or amputations.

# Let's get started! How to fit the Paww

This guide is here to help you confidently fit a Koalaa sleeve. Inside, you'll find clear, step-by-step instructions on how to correctly fit the Paww to a user, attach and swap out tools, and make simple adjustments to ensure a comfortable fit.



## Scan Me!

Scan the QR code to access helpful how-to videos for the Paww and other useful links!



## Need help? Reach out to our support team

At Koalaa, we're committed to making it easy for clinicians to get the support they need. Whether you have questions about measurements, fitting, ordering, or anything else related to our products and services, our team is here for you. Just get in touch, we're always here to help.

[hi@yourkoalaa.com](mailto:hi@yourkoalaa.com)

[+44 \(0\) 7482 165 098](tel:+44207482165098)

[@KoalaaCommunity](https://www.instagram.com/KoalaaCommunity)

[www.yourkoalaa.com](http://www.yourkoalaa.com)




## What you need to fit a Paww

Below is a list of everything you'll need to fit an Paww. Make sure you have all items ready before you begin, and follow the instructions step by step to guide you through the fitting process.



### Paww + Dock

Available in 3 sizes: This is the core structure of the Paww, with the dock securely attached at the palm

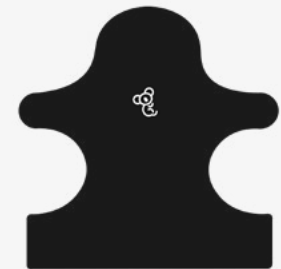

### Paww Cover

Available in 3 sizes: The cover adds style and an extra layer of protection



### Paww Tools

Choose from our range of functional tools to explore a wide variety of activities and tasks

What you will find in your Paww fitting kitt



**3x Paww Blanks**  
Small, Medium, Large


**3x Paww Covers**  
Small, Medium, Large

**5x Paww Tools**  
Janet, Amy, Rushton,  
Sarah, Grace

**3x Tool Foams**

Fitting Equipment

Measurements required for fitting the Paww

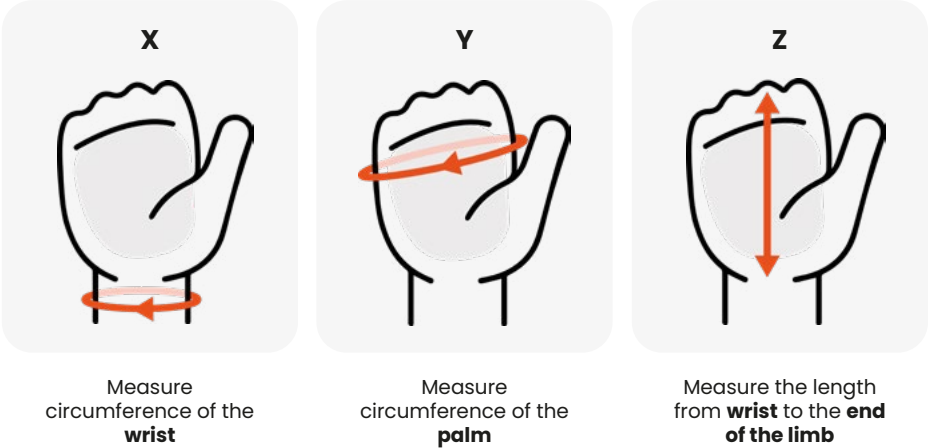

Paww Sizing Chart

Use measurement X, Y & Z to select the appropriate size

Measurement X	Measurement Y	Measurement Z	Paww size
7cm - 13cm	7cm - 14cm	3cm +	Small
12cm - 18cm	13cm - 20cm	5cm +	Medium
16cm - 22cm	17cm - 25cm	8cm +	Large



Scan for our online sizing calculator

Enter the user's limb measurements, and the calculator will determine the correct size Paww

## Fitting the Paww **blank**



Choose one of the three Paww sizes that best fits the user



Undo the Velcro straps and gently slide the user's palm into the Paww



Secure the Velcro straps around the palm and wrist, ensuring they're snug but comfortable



When you turn the user's hand over, ensure the dock is centred in the middle of the palm or limb

## Fitting the Paww **cover**



Choose the matching-size cover for the Paww



Place the cover on the back of the user's hand and secure it with the Velcro tabs



The top flap can be folded over for extra coverage if the user has a partial hand



For partial fingers, either fold the top of the cover inside



Or trim along the top to remove it



This will allow the digits to remain free for movement




## Fitting the Paww tools



Attach the tool to the dock by pressing it in firmly



The tool will stay in a fixed position

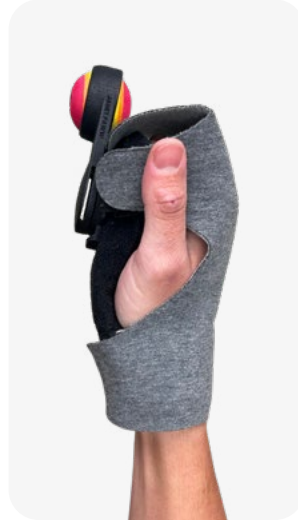

Keeping it secure on your palm for any activity



To release the tool, press the button underneath the dock



You can then rotate the tool and push it back into the dock to lock it



It can be set at any angle, a full 360°, anywhere around the dock

## Fitting the Paww tools



Easily swap out any of our tools and insert them into the Paww



Each is designed for a different activity



Simply click it in and go!

## Range of Paww tools

### The Janet



### The Amy



### The Rushton



### The Sarah



### The Grace



### The EDGA



### The Joanie



## How to care for the Paww

To keep the Paww in great condition and working at its best, a little care goes a long way. Below are some top tips to help keep it clean, comfortable.



Ensure the Paww is **comfortable** and not too tight on the user's arm. If the Paww leaves marks on the user's skin that don't fade within 20 minutes, it's too tight and should be loosened.



Make sure the Paww is positioned correctly, with the tool release button resting under the user's **palm or limb**.



Make sure all **digits** are clear of the dock when attaching tools to the Paww.



Regularly **clean** the Paww and tools by wiping away any excess dirt with a damp cloth to maintain hygiene and functionality.



To **wash** the Paww, remove the tool then place it in your regular wash at **30°C (86°F)**.



To extend the life of the Paww and tools, store them in a **cool, dry place** when not in use.

