



# How to use your Paww

A step-by-step guide on using the Paww for users with partial-hand or finger limb differences or amputations.

# Let's get started!

## How to use your Paww

This guide is here to help you get started with your Koalaa sleeve, so you can feel confident using it daily or for the activities you enjoy. Inside, you'll find simple, step-by-step instructions on how to put on your Paww, how to use and swap out the tools, and how to make small adjustments for your comfort.



### Scan Me!

Scan the QR code to access helpful how-to videos for your Paww and other useful links!



## Need help? Your limb buddy is here for you!

At Koalaa, there's no need for formal appointments or long waits, our limb buddies are always here and ready to help. Whether you need advice, support, or just a friendly chat, you can reach out anytime.

We're here to help with things like tool swaps, returns, or if something just doesn't feel quite right, just get in touch!

[hi@yourkoalaa.com](mailto:hi@yourkoalaa.com)

[+44 \(0\) 7482 165 098](tel:+44207482165098)

[@KoalaaCommunity](https://www.instagram.com/KoalaaCommunity)

[www.yourkoalaa.com](http://www.yourkoalaa.com)



### Putting on your Paww



Undo the velcro straps and slide your palm into the Paww



Do up the velcro straps so they're secure around your palm and wrist



When you turn your hand over, the dock should sit in the centre of your palm or limb



Add the cover to the back of your hand and use the velcro tabs to secure it on



The top flap can be folded over for extra coverage if you have a partial-hand



For partial fingers, to keep fingers free, you can either fold the top of the cover back or trim it off


## Adding your Paww tools



Attach the tool to the dock by pressing it in firmly



The tool will stay in a fixed position

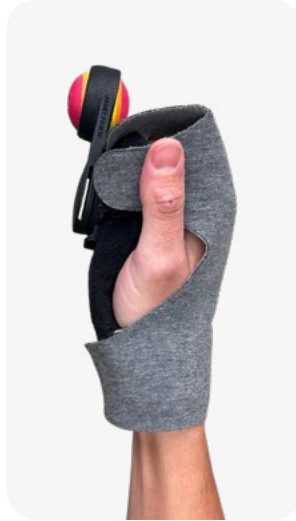


Keeping it secure on your palm for any activity



To release the tool, press the button underneath the dock




You can then rotate the tool and push it back into the dock to lock it

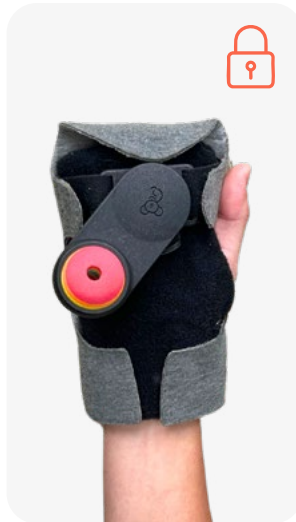


It can be set at any angle, a full 360°, anywhere around the dock

## How to use the **Amy Tool**

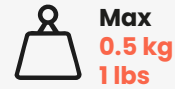

Max  
0.5 kg  
1 lbs

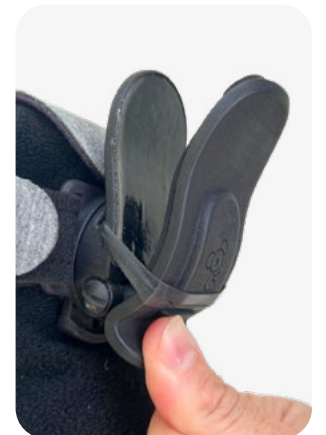






## How to use the **Rushton Tool**



 **Max**  
**10 kg**  
**22 lbs**



## How to use the **Janet Tool**

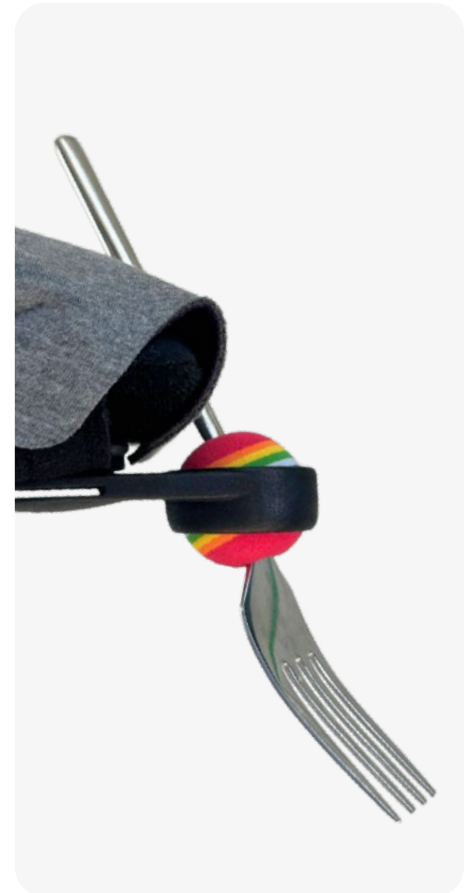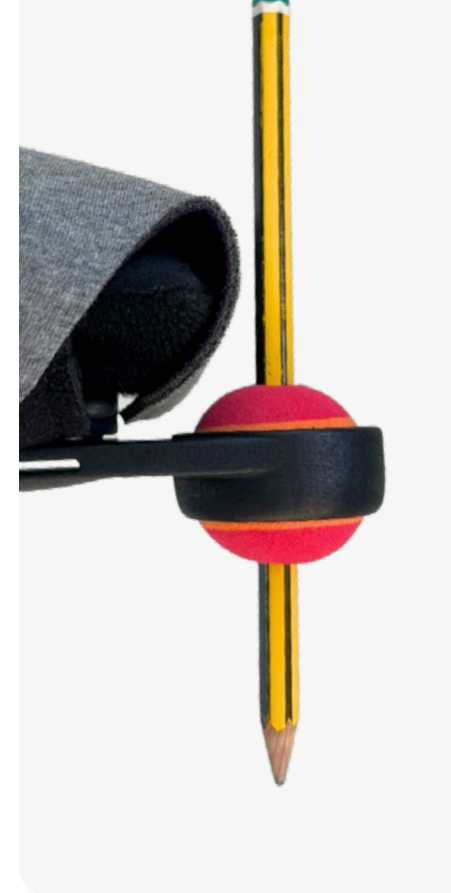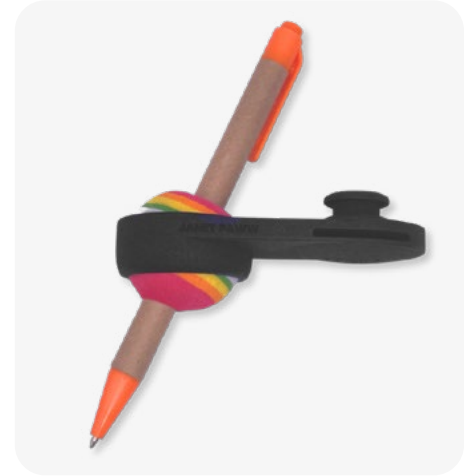



## How to use the **Sarah Tool**



 **Max**  
**10 kg**  
**22 lbs**



## How to use the **Grace Tool**

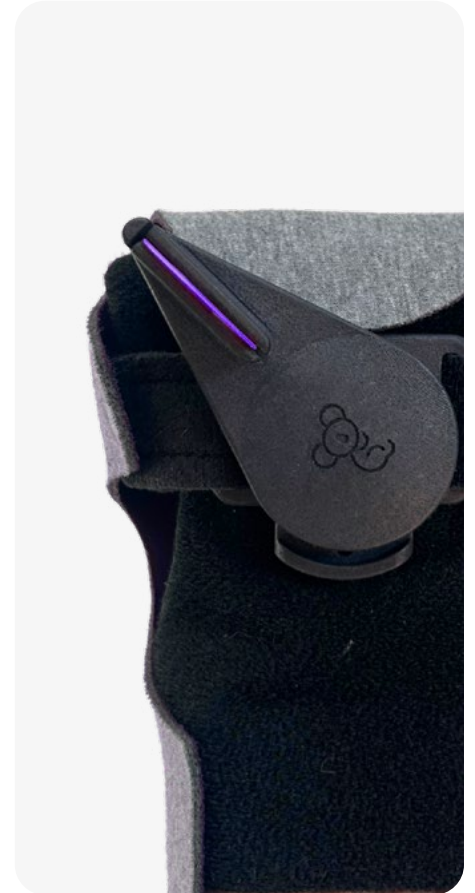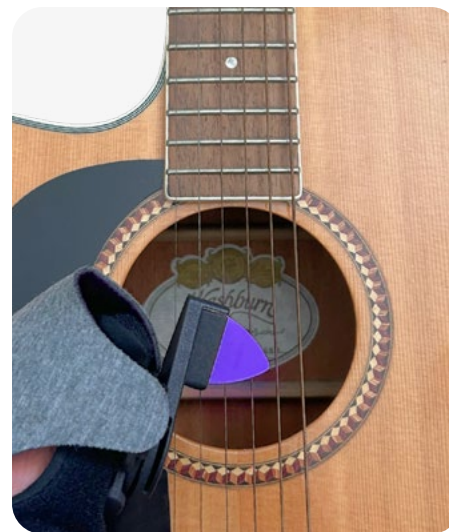

## How to use the **Joanie Tool**



## How to **care** for your Paww

To keep your Paww in great condition and working at its best, a little care goes a long way. Below are some top tips to help you keep it clean, comfortable, and ready for whatever you've got planned.



Ensure the Paww is **comfortable** and not too tight on your arm. If the Paww leaves marks on your skin that don't fade within 20 minutes, it's too tight and should be loosened.



Make sure the Paww is positioned correctly, with the tool release button resting under your **palm or limb**.



Make sure all digits are **clear** of the dock when attaching tools to the Paww.



Regularly **clean** your Paww and tools by wiping away any excess dirt with a damp cloth.



To **wash** your Paww, remove the tool then place it in your regular wash at **30°C (86°F)**.



To extend the life of the Paww and tools, store them in a **cool, dry place** when not in use.

