

Wunzi – a customized body hug that improves trunk alignment and stabilizes the core for those critical infant growth months.

Fase of Use

The elegant over-the-diaper design goes on in seconds. Closure tabs are embedded right onto the garments – no loose tabs to keep track of!

Familiar Design = Improved Compliance

Inspired by the standard infant "onesie" clothing, the Wunzi offers caregivers and clinicians a simpler alternative to the two garment TheraTogs system.

Three Systems in One

The Wunzi™ Infant System consists of the core-strengthening Wunzi garment set, plus size-appropriate limb cuffs and straps. It's a complete infant-sized alternative that combines the benefits of three systems - the Posture & Torso Alignment system, Full Body system, and Lower Extremity system – into one easy-to-use package.



Indications for Use Include:

Hypotonia • Diplegic or Hemiplegic • Cerebal Palsy • Torticollis • Athetosis

Cerebellar Ataxia • BPI • Developmental Delay



Babies build trunk and neck control and strength before they gain functional skills.

Trunk control problems that occur in infancy typically lead to long-term developmental deficits in limb use and movement. Postural control acquisition is a necessary component of early intervention.

The Wunzi system is designed to help very young clients by increasing sensory input while improving core stability. Like all TheraTogs systems, the Wunzi can support therapy goals both during and between therapy sessions, providing essential carry-over. In short: Wear it, live it, learn it!

- Beverly Cusick, PT, MS, COF | TheraTogs Inventor



Use the Wunzi to:

- ✓ Increase sensory awareness and information
- ✓ Train functioning trunk muscles in shorter state to improve postural alignment and control
- ✓ Limit excessive trunk motions
- ✓ Improve functioning alignment of hips and/or shoulders
- ✓ Gently stabilize one or both scapulae
- ✓ Improve respiration



Measurements for Selecting Wunzi Size

Torso Length (in CENTIMETERS) (see Fig. 1) - Measure down from C-7 over the diaper and up to the child's navel. Be careful to wrap the tape gently under the diaper. **Belly Girth** – Measure the infant's belly circumference at the navel (in cm).

(Fig. 1) Torso Length

Model No.	Model Name	Age	Weight	Torso Length	Belly Girth
GWZ25	Wunzi, XXSmall	0 – 3 mo.	3.2 – 5.4 kg	40 cm	40 cm
GWZ50	Wunzi, XSmall	3 – 6 mo.	5.4 – 7.7 kg	50 cm	44 cm
GWZ75	Wunzi, Small	6 – 12 mo.	7.7 – 8.2 kg	55 cm	46 cm
GWZLKS	Limb Kit, Small	0 – 12 mo.	Up to 8.2 kg		
GWZ100	Wunzi, Medium	12 - 18 mo.	8.2 - 9.5 kg	60 cm	50 cm
GWZ125	Wunzi, Large	18 – 24 mo.	9.5 – 11.8 kg	68 cm	54 cm
GWZLKL	Limb Kit, Large	12 – 24 mo.	Up to 11.8 kg		

Like all TheraTogs systems, the Wunzi is a comfortable, foam-lined, Velcro®-sensitive garment and strapping system that grips the skin. The patented composite fabric is elasticized with a stretch bias that favors good postural alignment – just the thing for a growing baby. The Wunzi includes grip-lined elastic straps that operate like external muscles and ligaments to improve functioning body and proximal joint alignment in order to optimize motor development.

How to Order

For ordering guidance visit www.TheraTogs.com/Ordering

About TheraTogs

TheraTogs™ are FDA Class I medical devices intended to be issued by, and applied under the supervision of, a licensed healthcare practitioner engaged in neuromotor or sensorimotor retraining. TheraTogs systems are made of GoldTone™ - a proprietary and patented composite fabric with foam backing made of an aqueous-based elastomeric urethane. TogRite™ strapping consists of elastomeric strapping with an inert, silicone-based grip surface. TheraTogs components are not made with natural rubber latex.