

Fitting Instructions

1. Remove inlay and place the A-Pace inside the shoe.
2. Place inlay over the top of the footplate. The inlay can be custom, off the shelf, or supplied with the user's footwear.
3. Pull the toggle towards the front of the shoe to open the magnetic buckles. Loosen the laces and slide your foot into the shoe. Fasten the buckles and adjust the strap using the end with the Trulife label. If you need to adjust the strap further, you can trim it at the other end.
4. Check fit:



HEEL STRIKE



MID STANCE




TOE OFF

- Perform trial walking as above. Ensure the knee is not hyperextended.
- Check footwear is not overly distorted.
- Ensure the patient is comfortable and there are no areas of pressure which may lead to lasting sores, or ulcerations.

5. Make adjustments if needed:

- The **footplate** can be trimmed with sharp scissors to fit a range of shoe sizes. Use the provided sanding sponge to smooth out any sharp edges.
- The **anterior shell** can be adjusted to the desired height using the provided hex key.

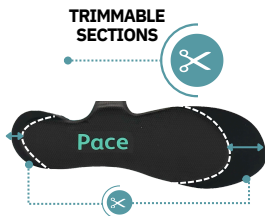

Intended use

- Pace products are designed to promote dynamic walking in patients with drop foot or other gait abnormalities.
- Pace products should be used only as directed by a medical practitioner.

Footwear selection

- For optimal performance and durability of the Pace range, footwear should have a firm heel counter, non-slip flexible sole, and enough room to accommodate the orthosis.
- Boots and high heels should not be worn.
- Pace products must not be used without footwear.

Maintenance

- The products should be regularly cleaned by wiping with a damp cloth and hand washing the removable pad and strap.



Important

- The product must NOT be used for running, squatting, kneeling, climbing ladders, or climbing stairs by loading the footplate unevenly.
- Discontinue use immediately if you notice any excessive skin redness, abrasion, or irritation.
- Any serious incident should be reported to the manufacturer and the relevant competent authority.

Disposal

- Please dispose of your A-Pace in accordance with local and national laws and regulations.

Warranty

- Hardware: 2 years
- Soft goods: 90 days



Trulife Limited
41 Amos Road, Sheffield
S9 1BX, United Kingdom

www.trulife.com



MDSS
Schiffgraben 41, 30175,
Hannover, Germany

