


QUICK-SIL™

50ml. Working Instructions

STEP ONE



Prepare Area

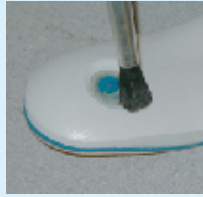


Brush Off
Dust

STEP TWO*



Shake Well

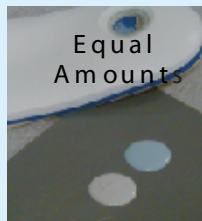

Equal
Amounts




Mix



Spread with
Cotton Swab

To adhere Quick-Sil to material, spread a small amount of QuickPrime over area. (*Not necessary if using silicone adhesive)

STEP THREE

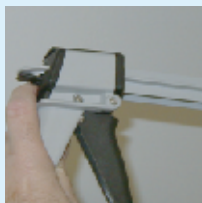

Set Up
Mixing Gun



Add Static
Mixer



Inject
Quick-Sil



Wait Five
Minutes

Follow the steps below to adhere Quick-Sil to materials using silicone adhesive:




Remove Set
Silicone



Apply thin coat
of silicone adhesive



Place Quick-Sil

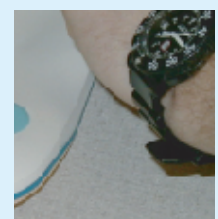

Wait at least
30 minutes