

# Xtend® Connect Shape Invisible Quick Connect/Disconnect

Manufacturer's  
recommended  
L-code  
5617

**Material**

Made of lightweight aluminum.

**Double security**

Double security locking  
function with two buttons.

**Patient weight**

Maximum patient weight is 220 lbs.

# Making day-to-day life easier

The patented Xtend Connect Shape is a prosthetic adapter designed to be installed within a cosmetic cover, though it functions just as effectively without one. It serves as a valuable aid for daily tasks, whether the user manages it independently or with the assistance of a caregiver.



## Getting dressed

Donning pants over a rigid prosthetic foot can be quite challenging, particularly with tighter pants. However, with the Xtend Connect Shape, you simply remove the foot, slip on the pants, and then reattach the foot effortlessly. Voilà!



## Change shoes

Switching from one shoe to another, or tying shoelaces comfortably, can be made easier with a simple hand grip. Just take off the foot and put it in your lap while you do the job. This method works well for folks who like to sit down while changing shoes or tying shoelaces, making it more comfortable, safer and convenient.



## Relax in comfort and style

Sometimes stiff and heavy prosthetic feet or legs makes it difficult to find a comfortable and relaxing position. Wherever you place the adapter, it's convenient to be able to disconnect parts at times. If the Xtend Connect Shape is placed above the knee, the user can easily take off the full weight of the leg when needed.



# It's so great you can't even see it!




A fully functional adapter in a leg with a cosmesis, without being visible. Stylish as well as functional.



## Requires minimal strength

The construction of Xtend Connect Shape makes it easy to handle. It can be effortlessly operated with a single-handed grip, but if that's not feasible, a two-handed push also works effectively.

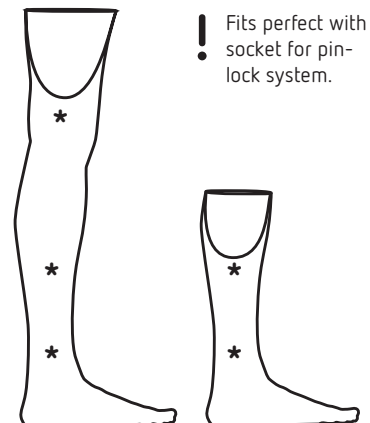


\* Possible installation points of Xtend Connect Shape.




# On the technical side

The Xtend Connect Shape consists of two main components - a quick fit unit and a base unit – that can be put in any direction you and your patient find suitable.

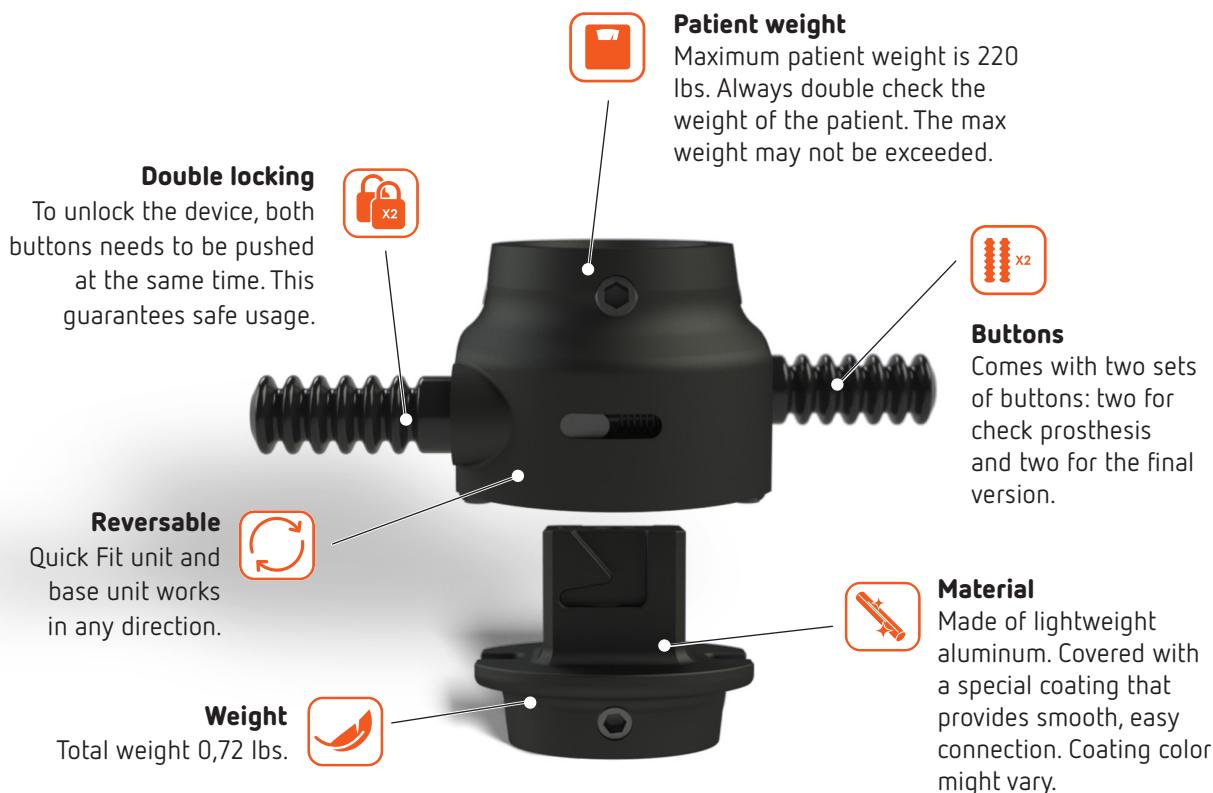The buttons for the locking mechanism can be cut to desired length when adapting to the thickness of the leg and cosmesis, in order to get the perfect fit.



## Xtend® Connect Shape's different components

	Item number	Description
	A200-SK	<b>Complete kit – ready to go:</b> Quick Fit unit and Base unit, both with female pyramid adapter.
	A200-SQF	<b>A la carte – advised if you need multiple units:</b> Quick Fit unit with a female pyramid adapter.
	A200-SBF	Base unit with a female pyramid adapter.

The warranty period is one year.

