



BIRD & CRONIN INC.

Reusable Therapeutic Cold or Hot Pack

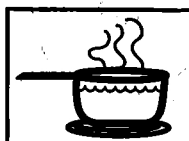


Bring water to boil.
Remove pan from heat.

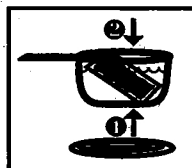


Place pack in water 8-10
min.



Do not apply pack
directly to skin.

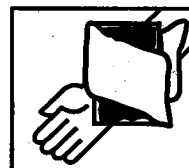

Wrap pack in cloth.

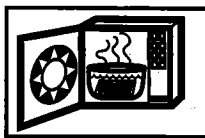


Cover pack with water in
microwave safe dish and
heat for 2 min.





Do not overheat.



Do not apply pack
directly to skin.

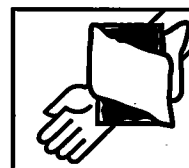


Wrap pack in cloth.

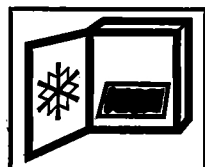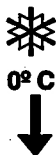

Place in freezer.



Do not apply pack
directly to skin.

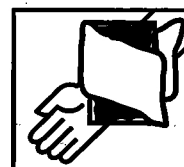

Wrap pack in cloth.

WARNING: Use only as directed. Always use a cloth cover between the gel pack and the skin to prevent frostbite or burns. Do not use on insensitive skin or on a person with poor blood circulation. Gel may be harmful if swallowed. If swallowed, dilute with water or milk and contact a physician. Discard pack if punctured or torn.

Non-Toxic

Distributed by:
Bird and Cronin, Inc
1200 Trapp Road, Eagan, MN 55121 USA

EU Authorized Rep.
(MDD93/42/EEC)
MDSS GmbH
Burkhardtstr. 1
D 30163



P920503