


# B-COOL™ Knee Wrap

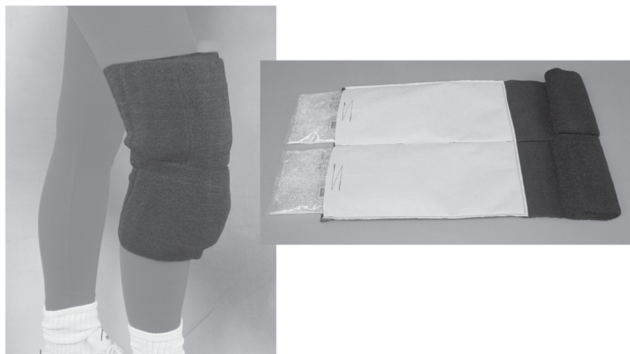

1. Remove wrap and insert gel packs into pockets.

*Note: Place gel packs into freezer until frozen*

2. Place wrap over affected knee.
3. Secure the attached straps around the knee to hold wrap in place.
4. Secure to wrap with hook tab.
5. Adjust straps accordingly for comfort.
6. Never apply a frozen or heated pack directly onto the skin.
7. Always follow your physicians or healthcare providers specific cold therapy treatment.

0814 3363 12 3/4"W x 11 3/4"H (32.4x29.9) Wrap Only  
0814 3361 12 3/4"W x 11 3/4"H (32.4x29.9) Wrap with B-Cool Gel 6" x 10" (12.6x25.4cm) 2/PK  
0814 3362 12 3/4"W x 11 3/4"H (32.4x29.9) Wrap with B-Cool 2.0 Gel 2/PK

## ***BIRD & CRONIN INC.***

*Designers and Manufacturers of Quality Medical Products*

1200 Trapp Road • Eagan, Minnesota USA 55121  
800/328-1095 • Fax 800/279-7934 • 651/683-1111 • Fax 651/688-9855  
email sales@birdcronin.com www.birdcronin.com 2009 10003363CM