

B-COOL™ Shoulder Wrap

Application Instructions



1. Remove wrap and insert gel packs into pockets.

Note: Place gel packs into freezer until frozen

2. Place wrap over affected shoulder.
- 3.. The attached strap wraps around the upper arm to secure in place. Attach with tab the longer strap to the wrap and pull around chest and secure back onto the wrap.
4. Hook tabs maybe adjusted for shoulder contour.
5. Adjust straps accordingly for comfort.
6. Never apply a frozen or heated pack diectly on the skin.
7. Always follow your physicians or healthcare providers specific cold therapy treatment.

Reorder No.	Dimensions	Description
0814 3323	12 3/4"W x 11 3/4"H (32.4 x 29.8cm)	Wrap Only
0814 3321	12 3/4"W x 11 3/4"H (32.4 x 29.8cm)	Wrap with B-Cool 6" x 10" Gel 2/PK
0814 3322	12 3/4"W x 11 3/4"H (32.4 x 29.8cm)	Wrap with B-Cool 2.0 Gel 2/PK

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products

1200 Trapp Road • Eagan, Minnesota USA 55121

651/683-1111 • Fax 651/688-9855 • 800/328-1095 • Fax 800/279-7934

e-mail sales@birdcronin.com • www.birdcronin.com

©2009 Bird & Cronin, Inc. All Rights Reserved -10003323