



Bicro™ Shoulder Immobilizer

Application Instructions


STEP 1 Secure immobilizer around chest area with cutout portion under arm to be immobilized. Contoured portion on female immobilizer should be positioned under breasts for patient comfort.

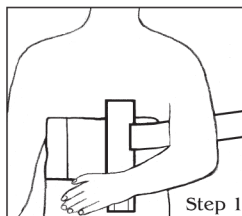

STEP 2 Place arm in humeral cuff and secure with hook and loop closure.

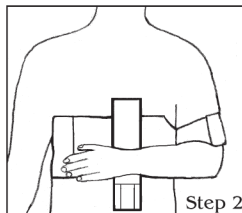

STEP 3 Place wrist in foam padded wrist cuff and secure with hook and loop closure. Slide hand through hand cuff to help avoid wrist drop.*

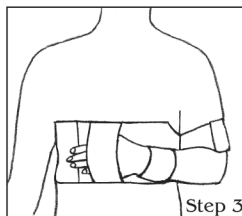

** Hand cuff is included on the straight immobilizer. This feature is not available on the contour version due to the contoured cutout.*

Contact your healthcare provider or therapist for further instruction.

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products

1200 Trapp Road • Eagan, Minnesota USA 55121

651/683-1111 • Fax 651/688-9855 • 800/328-1095 • Fax 800/279-7934

e-mail sales@birdcronin.com • www.birdcronin.com

©2000 Bird & Cronin, Inc. All Rights Reserved - Form 4500