

Tri-Panel Knee Immobilizer

Application Instructions

It is critical to determine the proper length of the knee immobilizer prior to the actual application. A correctly sized immobilizer fits on the leg 2" to 3" above the lateral malleolus and 6" to 8" below the femoral head.
5.1 - 7.5cm 15.2 - 20.3cm



Reorder No.	Size	Length	Width		CM Length	Width(CM)	
			Top	Bottom		Top	Bottom
0814 2741	Universal	12 1/2"	23 1/2"	19 1/2"	31.8cm	59.7cm	49.5cm
0814 2742	Universal	16"	23"	18 1/2"	40.6cm	66.cm	47.cm
0814 2743	Universal	18"	23"	19"	45.7cm	58.4cm	48.3cm
0814 2744	Universal	20"	23"	19 1/2"	50.8cm	58.4cm	49.5cm
0814 2745	Universal	22"	23"	18"	55.9cm	58.4cm	45.7cm
0814 2746	Universal	20" X-Wide	27 1/2"	21"	50.8cm	70.cm	53.3cm
0814 2747	Universal	24"	24"	18"	61.cm	61.cm	45.7cm
0814 2748	Universal	26"	23 1/2"	17 1/2"	66.cm	59.7cm	44.5cm



2. Place the posterior panel under the knee making sure the contour stay follows the popliteal area of the back of the knee. Secure immobilizer with "temporary secure strap" to hold in place temporarily while applying.

1. Separate 3 pieces.



3. A. Align side panel with the lateral side of the leg.

4. Secure hook and loop closure straps.

5. Optional patella strap may be attached over the patella.

B. Align 2nd side panel with the medial side of the leg.

Note: Number of straps vary depending on the length of immobilizer

NOTE This product should be used only under the supervision of a medical professional. It is not intended for unsupervised public use. If pain, swelling, changes in sensation or unusual reaction occur immediately contact your healthcare professional.

BIRD & CRONIN INC.

Designers and Manufacturers of Quality Medical Products

1200 Trapp Road • Eagan, Minnesota USA 55121

651/683-1111 • Fax 651/688-9855 • 800/328-1095 • Fax 800/279-7934

e-mail sales@birdcronin.com • www.birdcronin.com

© 2009 Bird & Cronin, Inc. All Rights Reserved - 10000633CM