Instructions for use

Icelock® Expulsion Valve 552

Product Number : L-552000

Life Without Limitations®

Ⓔ	EN	Instructions for use.....	2
Ⓔ	DE	Gebrauchsanweisung.....	5
Ⓔ	FR	Guide de fabrication.....	8
Ⓔ	ES	Instrucciones para el uso.....	11
Ⓔ	IT	Istruzioni per l'uso.....	14
Ⓔ	DA	Brugsanvisning.....	17
Ⓔ	SV	Bruksanvisning.....	20
Ⓔ	EL	Οδηγίες χρήσης.....	22
Ⓔ	NL	Gebruiksaanwijzing.....	25
Ⓔ	PT	Instruções de Utilização.....	28
Ⓔ	JA	取扱説明書.....	31
Ⓔ	ZH	中文说明书.....	34
Ⓔ	KO	사용 설명서.....	36

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ENGLISH

CONTENT

- Housing
- O-ring
- Retainer
- Valve
- Membrane
- Dummy for Rigid Sockets (RED)
- Dummy for Flexible Sockets (WHITE or GREEN)
- 2x Nails

TOOLS REQUIRED

- 2mm (1/16") Drill bits
- Hammer

CAST RECTIFICATION

- Determine the valve location on the plaster model and flatten the area with a Sureform to ensure the dummy sits flush. In the center hole of the dummy place a mark on the plaster model with an indelible marker.
- Drill a 2mm (1/16") vacuum hole using a long drill.
- Attach the dummy to the plaster model, using the nails provided.

IMPORTANT:

For rigid sockets choose RED dummy.

For flexible inner sockets choose WHITE or GREEN dummy.

PREPARATION FOR THERMOPLASTIC SOCKETS

- Drape the thermoplastic socket.
- Guide the plastic around dummy to ensure proper molding.

PREPARATION FOR LAMINATED SOCKETS

- Place the first PVA bag over the plaster model with dummy in place.
- Place the reinforcing layers and second PVA bag over the dummy. After mixing the resin apply the vacuum to remove trapped air. Pour the resin.

FINISHING AND INSTALLATION

- Expose dummy by grinding away material until fully flat area is exposed.
- Remove dummy and insert housing with O-ring.
- Firmly hand tighten the retainer to the housing. Make sure the O-ring is not visible from inside the socket.

DISASSEMBLY AND CLEANING

- Unscrew valve and remove the membrane.
- Clean under running water with mild detergent and clear airways with an air hose. Hold down the button in the cap while clearing airway.

REASSEMBLY

- Replace the membrane with convex surface facing down.

PRODUCT WARRANTY

- OSSUR offers 6 months limited warranty on manufacturing faults.

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