Instructions for use

Miami JTO® Thoracic Extension

Life Without Limitations®

		4
(EN)	Instructions for use	6
(DE)	Gebrauchsanweisung	7
(FR)	Notice d'utilisation	9
(ES)	Instrucciones para el uso	13
(IT)	Istruzioni per l'uso	15
(NO)	Bruksanvisning	17
(DA)	Brugsanvisning	19
(SV)	Bruksanvisning	21
(FI)	Käyttöohjeet	23
(NL)	Gebruiksaanwijzing	25
(PT)	Instruções de Utilização	27

EN - Caution: This product has been designed and tested as a single use product, and is not intended for multiple use. In addition to increasing the risk of infection, used product components may contain small unnoticeable defects that may lead to non-optimal performance, and even mechanical failure. If any problems occur with the use of this product, immediately contact your medical professional.

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300
Small

PURPLE

Does your patient have a kyphotic (chin-on-chest) neck?

- Kyphotic
- Chin-on-chest.
- Geriatric
- Osteoporitic and/or ankylosing spondylitic

NAVY

Does your patient have a very large neck circumference?

- Very large neck circumference
- Obese
- "No-Neck".
- Massive shoulders.

GREEN

Does your patient have a kyphotic (chin-on-chest) neck?

- Very short, thin neck.
- Principally female.
- More prevalent in the Asian population.
- Rare.

LT. BLUE

Does your patient have a long, tall neck?

- Long, tall neck.
- "Swan neck".
- Young women.
- Adolescents.

ORANGE

Is your patient female?

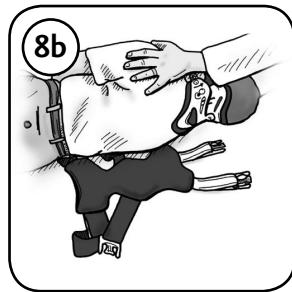
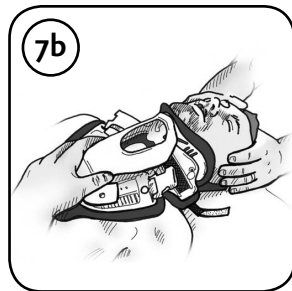
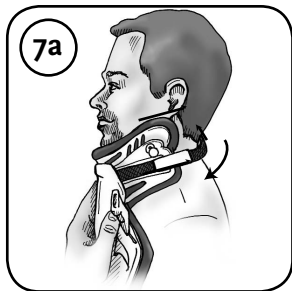
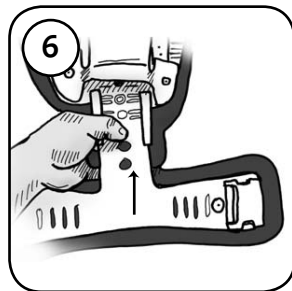
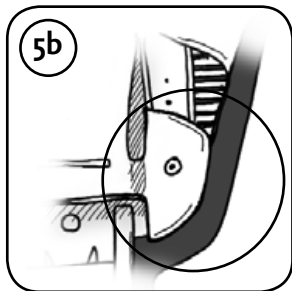
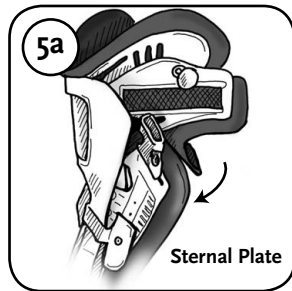
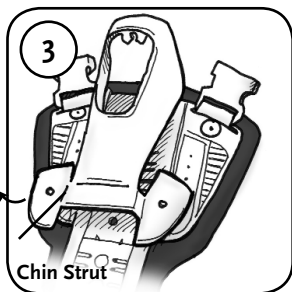
- Female
- Standard size for mature women.
- Thin, mature men.

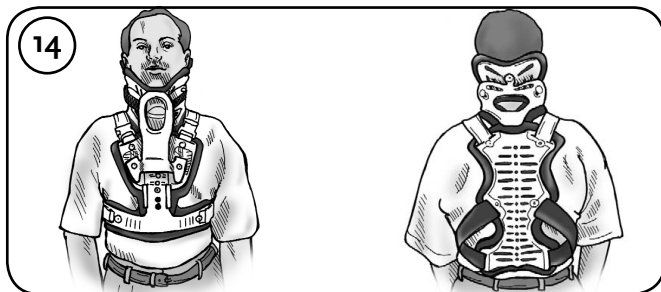
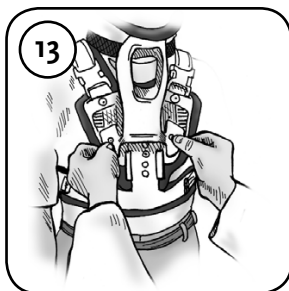
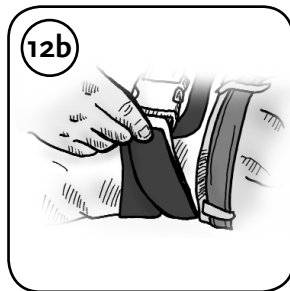
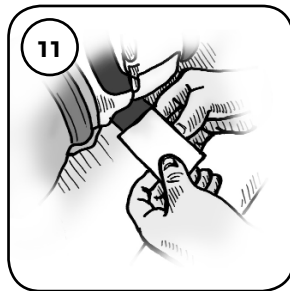
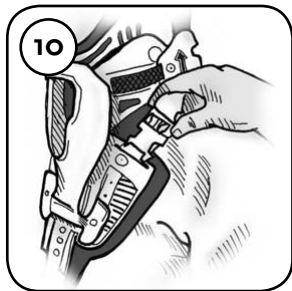
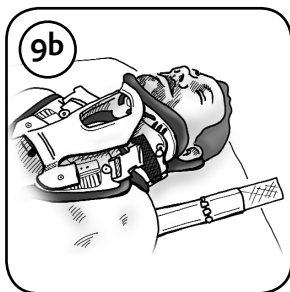
YELLOW

All other adults.

- Standard size for mature men.
- Ample or swollen women.

ENGLISH

The Miami JTO® is designed exclusively for use with the Miami J® Collar. Fits adult patients with waist circumference greater than 24 inches (61 cm). This is a Latex-Free product. For instructions on proper care and cleaning of the Miami JTO® and collar, please consult the Miami JTO® Patient Handbook.

Patients under C-Spine precautions require supine application of the Miami JTO®.

PRE-APPLICATION OF MIAMI JTO

1. Select appropriate size Miami J® Collar (see Miami J® Sizing & Application Instructions or Wall Chart). The Miami JTO may be applied over light clothing (**Figure 1**).
2. Remove blue and white sternal pad from collar (**Figure 2**).
3. Remove Chin Strut from Miami JTO Front Piece (**Figure 3**).
4. Squeeze collar to expose chin rivet. Attach Chin Strut. The top inside opening of the Chin Strut will slide around the center rivet on the collar and snap into place (**Figure 4**).
5. A) Attach Chin Strut feet to the Chest Plate in the lowest possible position (height may be adjusted later). The Sternal Plate on the collar attaches to Velcro® at the top of the Miami JTO Chest Plate. Make sure the blue pad of collar is behind the Chest Plate pad (**Figure 5a**).
B) Rivet holes in the Chin Strut can be aligned with corresponding holes on the Chest Plate to insure symmetrical positioning (**Figure 5b**).
6. If not already attached, attach Miami JTO Front to Chest Plate, using one of the two bottom openings. The Miami JTO Front should be set at the highest possible position and should be above the costal margin (**Figure 6**).

MIAMI JTO APPLICATION - UPRIGHT / SUPINE

- A) Use Retaining Strap to loosely apply front of Miami JTO to patient. Sides of the collar Front Piece should be oriented up, off the trapezius, and toward the ears (**Figure 7a**).
- B) SUPINE APPLICATION: Ensure patient's head is in treatment position, and is stabilized (**Figure 7b**).
7. A) Center collar Back Piece on patient's neck and attach Velcro straps to front of collar. Tighten straps one at a time, to an equal length on both sides. Velcro straps should be aligned "blue-on-blue."
- Collar application MUST BE secure to ensure proper fit and maintain alignment. Optional: Attach 4" Shoulder Pads (in Accessories Pack) to shoulder straps of Miami JTO Back Piece. Proper positioning of the Back Piece will allow scapulae to move freely (**Figure 8a**).
- 8b. SUPINE APPLICATION: Logroll patient onto side. Place the Miami JTO Back Piece on the bed beside the patient's back, with shoulder and waist straps spread out (**Figure 8b**).
8. 9a. Position the Miami JTO Back Piece to rest in the center of the patient's back. The top edge should rest approximately at T1/T2 (**Figure 9a**).
9. 9b. Return the patient to the supine position (**Figure 9b**).
10. Attach the Miami JTO Front to the Back using the shoulder buckles. Adjust the shoulder straps for a snug fit. Both shoulder straps should be of equal length, and Front and Back pieces should be centered on the patient (**Figure 10**).
10. 11. Remove the white double-faced Velcro strips from the Waist Belts.

11. Do not discard! Optional: Loosen straps, and attach 8" Waist Pads (in Accessories Pack) **(Figure 11)**.
12. A) Loosely buckle Waist Belts. Pull straps even and tight **(Figure 12a)**.
B) **Fasten Waist Straps with white doublefaced Velcro strips. Make sure all straps lie flat with no slack or gaps. Trim excess waist strap and indicate proper placement position (Figure 12b).**
13. Adjust height of Chin Strut to comfortably fit under patient's chin with head in treatment position. The Chin Strut should normally be set at the lowest position on the Velcro. The strut height may be raised for use with the Super Short collar and/or for positioning the patient in extension **(Figure 13)**.
14. Adjust all straps as necessary. When properly fit, you should not be able to freely insert a finger behind the top of the Chest Piece.

FINAL CHECKLIST (FIGURE 14)

- Chin is centered comfortably in the chin piece.
- Collar Velcro straps oriented blue-on-blue, both tabs the same length.
- Front of collar angled up toward ears.
- Front and Back Pieces of Miami JTO are centered on the body.
- No plastic touching the skin.
- No slack or gaps in any of the straps.
- Chin Strut set at lowest permissible treatment position.
- Lower edge of Front Piece resting against rib cage and/or just below breasts.

ADJUSTMENTS

- Excess Pad and Belt material may be trimmed. For best • appearance, Pad material may be trimmed with dull scissors to maintain a smooth seam.
- Proper positioning of Miami JTO Front is just below the clavicle, with the
- Front Piece no lower than the costal margin (particularly with a distended abdomen). The height of the Front Piece may be adjusted to accommodate patient anatomy. The primary indication for lengthening the Front Piece is to accommodate large breasts.
- If there is too much extension, or pressure on the chin, lower the Chin Strut height slightly.
- Snap rivets (in Accessories Pack) may be used to permanently attach Chin Strut to Chest Plate and lock height adjustment mechanism.
- The Chin Strut may be removed **(with physician approval)** for sleep or when the patient is supine.

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Össur Iberia S.A.