

## RCAI Basic LSO 452LSO



The new **Basic LSO** by RCAI is indicated for chronic back pain, post op rehab, chronic lumbar instability, herniated disc, or sprains and strains of the lower back. Rigid, heat moldable Kydex<sup>®</sup> back panel insert allows for patient customization and lumbar stability. Latex free side pulls provide therapeutic compression and added support.




Back view

### SIZING

	Measure Hip Circ.	Model #
Small	33 – 38" (83.8 - 96.5 cm)	452LSO-S
Medium	38 – 43" (96.5 - 109.2 cm)	452LSO-M
Large	43 – 48" (109.2 - 121.9 cm)	452LSO-L
XLarge	48 – 53" (121.9 - 134.6 cm)	452LSO-XL




Limited Warranty Coverage: Any defect in material or workmanship solely limited to the Kydex<sup>®</sup> plastic frame or metal portion or hinge mechanisms for 1 (one) year from date of purchase.

## APPLICATION

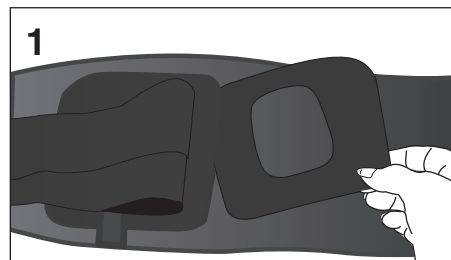

Heat and mold posterior panel to desired lordosis.

Open the LSO and align so the anterior (front) and posterior (back) panels are centered on your body. Wrap left side panel around to front and secure with hook and loop material provided.

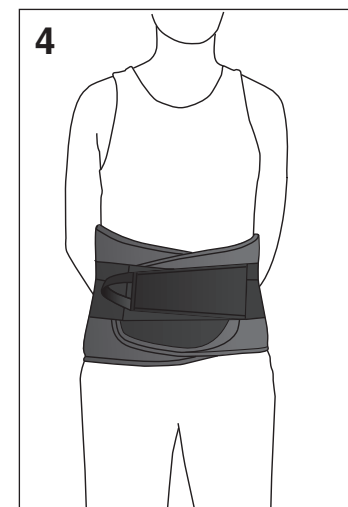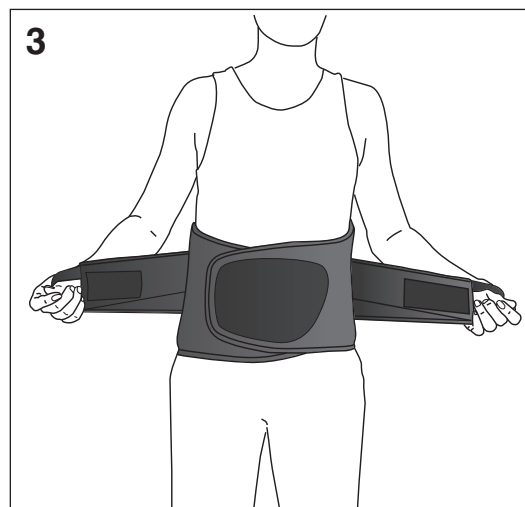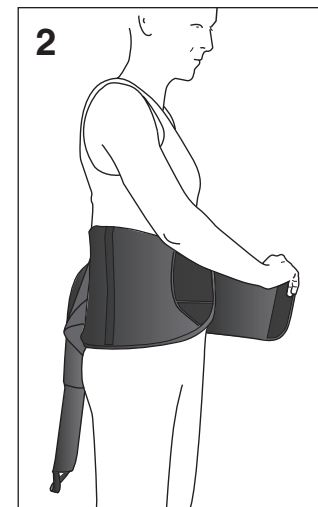

Reach back with both hands for the elastic side pulls on each side of the brace. Bring the left and right pulls forward and secure to the front of the LSO by attaching one side over the other.