



ECLIPSE

Suspension Sleeve Instructions



The Eclipse is a unique two seal suspension system for the active amputee. The Eclipse is designed to be used in variable setups.



The outer Eclipse sleeve can be rolled or folded down, using only the inner sleeve to suspend. For best suspension, use both sleeves in conjunction with each other. Many variations are being employed by the amputees currently wearing the Eclipse.



Shiny strip of silicone

Donning the Eclipse



1. Don the short inner sleeve on a clean socket low enough that the shiny end is completely sealed against the socket wall. Be sure the proximal end of the inner sleeve covers the medial and lateral ears.

Shiny strip of silicone



2. Don the outer sleeve so that the edge of the outer sleeve is in line with the edge of the inner sleeve. Make sure the shiny strip of silicone of the outer sleeve is touching the inner sleeve's shiny silicone. Once in position roll the outer and inner sleeve down.



3. Don the prosthesis.



4. Pull up the inner sleeve.



5. Roll up the outer sleeve. Use extra care to force out any trapped air.



Durability suggestions:

1. Trim the socket edges so there is no extra reverse flairs.
2. Carbon sockets need a coating (super glue) applied to all edges. Carbon is a strong abrasive.
3. Roll the prosthetic socks over the ears before donning the inner Eclipse sleeve.
4. When a hole occurs rotate the outer or inner Eclipse sleeve 5 degrees. The rotation will seal the leak.

