

Active Reliever

Osteoarthritis Knee Brace



Keep patients more active with less pain.

After 20 years of market leadership in rigid OA bracing, Townsend's Active Reliever features our decompressive offloading technology in a lightweight, single upright design.



Medical professionals can contour the aluminum bars to customize the fit and counterforce




Integrated semi-rigid nylon thigh and tibia shells enhance stabilization, correction and anti-rotation



Adjustable Corrective Force Strap can be tensioned to optimize the patient's pain relief



Adaptable patella buttress pad

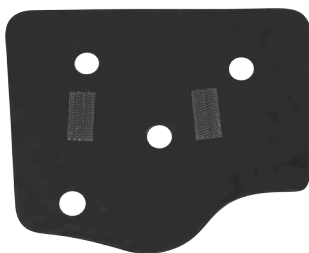

Interior thigh panel can be removed and trimmed for custom fitting



Ergonomically designed quick release buckles make it easier for the patient to apply and remove the brace

*Highlights in red demonstrate unique features

Indications For Use...

The Active Reliever is a decompressive single upright OA knee brace designed to reduce load on one side of the knee that has been injured or progressively degenerated. The Active Reliever should be dispensed by qualified medical professionals based on a physician's prescription and directives. Clinical conditions that can be treated by this brace include:

Routinely Prescribed For...

- Unicompartment Osteoarthritis
- Articular Cartilage Defect Repair
- Meniscal Cartilage Repair
- Osteochondral Defect Repair

This Product Can Be Ordered...

- X-Small to XXX-Large
- Medial or Lateral OA
- Left or Right

Product Features...

- Breathable spacer fabric sleeve with posterior expandable Lycra panel
- Anatomically correct roll-and-glide Townsend Motion Hinge with adjustable ROM stops
- Semi-rigid, self-contouring nylon superstructure sewn into the sleeve
- Bendable bars for customization of the fit and counterforce
- Corrective Force Strap applies maximum counterforce at midstance phase of gait
- Patient friendly quick release buckles
- Open anterior thigh section for easy donning and doffing

Caution

Specific instructions about the safe use of this product for general lifestyle activities should be provided by the prescribing physician or other medical professional.

Instruct the patient to **IMMEDIATELY DISCONTINUE** use of the brace and contact the medical professional who provided or fit the brace if the patient experiences:

- Abnormal redness that lasts more than 10 minutes after removing the brace
- Skin irritation, bruising, blistering or abrasion
- Any numbness, tingling or other indication of a loss of circulation to your leg or foot.

Actual patient results can vary without proper fitting, application and use of the product.



Ordered by: _____ Phone # (_____) _____ Patient's Name: _____

BILLING: P.O. Number _____ Townsend Account # _____

Bill To: _____

Ship To: _____

Address: _____

Address: _____

City: _____

City: _____

State: _____ Zip Code: _____ Country: _____

State: _____ Zip Code: _____ Country: _____

Phone: (_____) _____ Fax: (_____) _____

Phone: (_____) _____ Fax: (_____) _____

Shipping Preference: ☐ Ground ☐ 2-Day P.M. ☐ 2-Day A.M. ☐ Next Day P.M. ☐ Next Day A.M.
(If no preference is indicated, this order will be shipped 2 Day P.M.) Note: We do not ship products directly to patients.

The brace you order is determined by the leg, affected knee compartment, and size.

The Active Reliever features a universal (left or right leg) sleeve. The brace can be ordered with the hinge mounted on the left side of the sleeve or the right side of the sleeve. **THE HINGE SHOULD ALWAYS BE ON THE INJURED/DAMAGED SIDE OF THE KNEE.**

☐ Left Leg Medial OA / Right Leg Lateral OA

☐ Right Leg Medial OA / Left Leg Lateral OA

Size

☐ X-Small ☐ Small ☐ Medium ☐ Large

☐ X-Large ☐ XX-Large ☐ XXX-Large

Optional

☐ Flexion Stop Kit

Sizing Reference

(6" above mid patella)

X-Small	13" to 15"
Small	15" to 18"
Medium	18" to 21"
Large	21" to 23"
X-Large	23" to 26"
XX-Large	26" to 29"
XXX-Large	29" to 32"

Townsend Design

4615 Shepard St., Bakersfield, CA, 93313
Phone: 800.432.3466 or 661.837.1795; Fax: 800.798.2722

Please complete and fax this form to 800.798.2722 (24-hours a day). If you are calling in your order, this form indicates the options and information that will be required by our staff. For phone orders, please call 800.700.2722 between 7:00 a.m. and 4:00 p.m. (PST).